ECOCart

DE Salze für saures Dialysekonzentrat – **FR** Sels pour concentré acide d'hémodialyse – **IT** Sali per concentrato acido dialisi – **EN** Salts for acidic dialysis concentrate – **ES** Sales para concentrado ácido para diálisis – **PT** Sais para concentrado ácido de diálise – **FI** Suolat happamalle dialyysikonstraatille – **NL** Zouten voor zuur dialysecontraat – **SV** Salter för surt dialyskoncentrat – **DA** Salte til sur dialysekoncentrat – **HU** Sók a savas dialízis koncentrátumhoz – **CS** Soli pro kyselý hemodialyzační koncentrát – **SK** Soli pre kyslý hemodialyzačný koncentrát – **SL** Soli za kislinski dializni koncentrat – **RU** Соли для кислого диализного концентрата – **KK** Қышқыл диализ концентратына арналған тұздар – **UK** Солі для кислотного концентрату діалізу – **RO** Sare pentru concentrat de dializă acidă – **PL** Sole do kwaśnego koncentratu dializacyjnego – **HR** Soli za koncentraciju kiselog dijalize – **ZH** 酸性透析浓缩液的盐 – **VI** Muối cô đặc thấ́m phân axit – **EL** Αλάτια για συμπύκνωση όξινης διαλύσεως – **NO** Salter for sure dialysekoncentrat – **BG** Соли за кисела диализа концентрат – **ET** Happelise dialüüsi kontsentraadi soolad – **LV** Skābes dialīzes koncentrāta sāļi – **TR** Asidik diyaliz konsantresi tuzları – **TH** เกลือสำหรับการฟอกไตที่เป็นกรดเข้มข้น – **LT** Druskos rūgštiniam hemodializės koncentratui

DE Gebrauchsanweisung – **FR** Instructions d'utilisation – **IT** Istruzioni d'uso – **EN** Instructions of Use – **ES** Instrucciones de uso – **PT** Instruções de Uso – **FI** Käyttöohjeet – **NL** Instructies voor gebruik – **SV** Bruksanvisning – **DA** Brugsanvisning – **HU** Használati utasítások – **CS** Návod k použití – **SK** Návod na použitie – **SL** Navodila za uporabo – **RU** Инструкции по использованию – **KK** Пайдалану нұсқаулары – **UK** Інструкція з експлуатації – **RO** Instrucțiuni de utilizare – **PL** Instrukcja użytkowania – **HR** Upute za uporabu – **ZH** 使用说明 – **VI** Hướng dẫn sử dụng – **EL** Οδηγίες Χρήσης – **NO** Bruksanvisning – **BG** Инструкции за употреба – **ET** Kasutusjuhised – **LV** Lietošanas instrukcijas – **TR** Kullanım Talimatları – **TH** คำแนะนำในการใช้ – **LT** Naudojimo informacija

DE Erklärung der Symbole umseitig – **FR** Explication des symboles au verso – **IT** Spiegazione dei simboli sul retro – **EN** Explanation of symbols overleaf – **ES** Explicación de símbolos al dorso – **PT** Explicação dos símbolos no verso – **FI** Symboli selitetään kääntöpuolella – **NL** Verklaring van symbolen op de volgende pagina – **SV** Förklaring av symboler på nästa sida – **DA** Forklaring af symboler på bagsiden – **HU** A szimbólumok magyarázata a túloldalón – **CS** Vysvětlení symbolů na druhé straně – **SK** Vysvetlenie symbolov na druhej strane – **SL** Pojasnilo simbolov na sliki – **RU** Объяснение символов на обороте – **KK** таңбалардың үстіңгі жағындағы түсіндірме – **UK** Пояснення символів на звороті – **RO** Explicația simbolurilor de pe verso – **PL** objaśnienie symboli na odwrocie – **HR** Objašnjenje simbola na poleđini – **ZH** 背面符号说明 – **VI** Giải thích các ký hiệu bên lậ – **EL** Επεξήγηση των συμβόλων στην επόμενη σελίδα – **NO** Forklaring av symbolene på baksiden – **BG** Обяснение на символите на обратната страна – **ET** Sümbolite selgitus järgmisel leheküljel – **LV** Simbolių paaiškinimas kitoje pusėje – **TR** Arkada sembollerin açıklaması – **TH** คำอธิบายสัญลักษณ์ด้านหลัง – **LT** Simbolių paaiškinimą žr. antroje lapo pusėje

MADE IN GERMANY

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Salts for acidic haemodialysis concentrate Medical Device

Information for use

Store frost-free. Avoid heat and direct sunlight. Store between 4°C and 35°C.

Keep out of reach of children. Use only clear solutions. Only for use with the ECOMix production system. Only to be used by trained dialysis personnel. The information, warnings and pieces of advice in the operating instructions of the ECOMix are part of this information and must be observed. All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is located.

Use only original sealed and undamaged containers. Do not use after the expiry date.

Open container only at the proper connection points and connect with production system.

Attention! When opening the couplings acetic acid vapours can escape. Acid may leak from damaged ECOCart. Treat damaged ECOCart with caution. For the electrolyte concentration, please refer to the product label.

Add a defined amount of purified water to produce an acidic dialysis concentrate designed for extracorporeal bicarbonate haemodialysis, or bicarbonate haemodiafiltration. The ECOCart is not intended for home dialysis. Water for the dissolution of the ECOCart must adhere to the specifications of the ISO 23500-3. The dialysis concentrate produced is intended for use with an alkaline bicarbonate haemodialysis concentrate (NaHCO₃) for a given dilution.

The finished haemodialysis solution must be tested before use using a suitable method. For safe use, observe the operating instructions of the manufacturer of the haemodialysis machines.

Acidic and alkaline bicarbonate haemodialysis concentrate should only be diluted immediately before use.

Duration and frequency of treatment as well as dosage must be defined by the physician.

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