ECOCart

DE Salze für saures Dialysekonzentrat – **FR** Sels pour concentré acide d'hémodialyse – **IT** Sali per concentrato acido dialisi – **EN** Salts for acidic dialysis concentrate – **ES** Sales para concentrado ácido para diálisis – **PT** Sais para concentrado ácido de diálise – **FI** Suolat happamalle dialyysikonstraattille – **NL** Zouten voor zuur dialyseconcentraat – **SV** Salter för surt dialyskoncentrat – **DA** Salte til sur dialysekoncentrat – **HU** Sók a savas dialízis koncentrátumhoz – **CS** Soli pro kyselý hemodialyzační koncentrát – **SK** Soli pre kyslý hemodialyzačný koncentrát – **SL** Soli za kislinski dializni koncentrat – **RU** Соли для кислого диализного концентрата – **KK** Қышқыл диализ концентратына арналған тұздар – **UK** Солі для кислотного концентрату діалізу – **RO** Sare pentru concentrat de dializă acidă – **PL** Sole do kwaśnego koncentratu dializacyjnego – **HR** Soli za koncentraciju kiselog dijalize – **ZH** 酸性透析浓缩液的盐 – **VI** Muối cô đặc thắm phân axit – **EL** Αλάτια για συμπύκνωση όξινης διαπίδυσης – **NO** Salter for sure dialysekoncentrat – **BG** Соли за кисела диализа концентрат – **ET** Happelise dialüüsi kontsentradi soolad – **LV** Skābes dialīzes koncentrāta sāļi – **TR** Asidik diyaliz konsantresi tuzları – **TH** เกลือสำหรับการฟอกไตที่เป็นกรดเข้มข้น – **LT** Druskos rūgštiniam hemodializės koncentratui

DE Gebrauchsanweisung – **FR** Instructions d'utilisation – **IT** Istruzioni d'uso – **EN** Instructions of Use – **ES** Instrucciones de uso – **PT** Instruções de Uso – **FI** Käyttöohjeet – **NL** Instructies voor gebruik – **SV** Bruksanvisning – **DA** Brugsanvisning – **HU** Használati utasítások – **CS** Návod k použití – **SK** Návod na použitie – **SL** Navodila za uporabo – **RU** Инструкции по использованию – **KK** Пайдалану нұсқаулары – **UK** Інструкція з експлуатації – **RO** Instrucțiuni de utilizare – **PL** Instrukcja użytkowania – **HR** Upute za uporabu – **ZH** 使用说明 – **VI** Hướng dẫn sử dụng – **EL** Οδηγίες Χρήσης – **NO** Bruksanvisning – **BG** Инструкции за употреба – **ET** Kasutusjuhised – **LV** Lietošanas instrukcijas – **TR** Kullanım Talimatları – **TH** คำแนะนำในการใช้ – **LT** Naudojimo informacija

DE Erklärung der Symbole umseitig – **FR** Explication des symboles au verso – **IT** Spiegazione dei simboli sul retro – **EN** Explanation of symbols overleaf – **ES** Explicación de símbolos al dorso – **PT** Explicação dos símbolos no verso – **FI** Symboli selitetään kääntöpuolella – **NL** Verklaring van symbolen op de volgende pagina – **SV** Förklaring av symboler på nästa sida – **DA** Forklaring af symboler på bagsiden – **HU** A szimbólumok magyarázata a túldoldalon – **CS** Vysvětlení symbolů na druhé straně – **SK** Vysvetlenie symbolov na druhej strane – **SL** Pojasnilo simbolov na sliki – **RU** Объяснение символов на обороте – **KK** таңбалардың үстіңгі жағындағы түсіндірме – **UK** Пояснення символів на звороті – **RO** Explicația simbolurilor de pe verso – **PL** objaśnienie symboli na odwrocie – **HR** objašnjenje simbola na poledini – **ZH** 背面符号说明 – **VI** Giải thích các ký hiệu bên lề – **EL** Επεξήγηση των συμβόλων στην επόμενη σελίδα – **NO** Forklaring av symbolene på baksiden – **BG** Обяснение на символите на обратната страна – **ET** Sümbolite selgitus järgmisel leheküljel – **LV** Simbolių paaiškinimas kitoje pusėje – **TR** Arkada sembollerin açıklaması – **TH** คำอธิบายสัญลักษณ์ด้านหลัง – **LT** Simbolių paaiškinimą žr. antroje lapo pusėje

MADE IN GERMANY

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Salts for acidic haemodialysis concentrate

Medical Device

Information for use

Store frost-free. Avoid heat and direct sunlight. Store between 4°C and 35°C.

Keep out of reach of children. Use only clear, colorless solutions.

For use with the ECOMix production system only. Use by trained dialysis personnel only. The information, warnings, and instructions in the ECOMix user manual are part of this information and must be followed. Any serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is located.

Use only unopened and undamaged containers. Do not use after the expiration date.

Open containers only at the designated connection points and connect them to the production system. Water for dissolving the ECOCart must comply with the requirements of ISO 23500-3.

Caution! Acetic acid fumes may escape when opening the couplings. Acid may leak from damaged ECOCards. Handle damaged ECOCards with care.

The electrolyte concentration can be found on the product label.

Intended Use

ECOCart is designed to be dissolved in the ECOMix in a fully automated process to produce an acidic dialysis concentrate. Dialysis concentrates prepared with ECOCart are suitable for hemodialysis and hemodiafiltration. The ECOCart is not intended for use in home dialysis.

Areas of application

The dialysis concentrate is intended for use with an alkaline bicarbonate hemodialysis concentrate (NaHCO_3) at a given dilution. The dilution instructions can be found on the ECOCart product label. The prepared hemodialysis solution must be tested using a suitable method before use. For safe use, refer to the operating instructions provided by the hemodialysis machine manufacturer. Acidic and alkaline bicarbonate hemodialysis concentrate should only be diluted immediately before use.

The following indications apply only to the ready-to-use hemodialysis solution prepared according to the instructions on the ECOCart:

- Acute kidney failure
- Chronic kidney failure
- Overwatering
- Poisonings
- Correction of acid-base balance and electrolyte status
- Settings of blood or plasma and body temperature

Contraindications

- Hyperkalemia when using hemodialysis concentrate with an excessively high potassium content
- Hypokalemia when using hemodialysis concentrate with too low a potassium content
- Uncontrollable coagulation disorders

Side effects

- Side effects such as hypotension, nausea, vomiting and cramps are possible during hemodialysis.

The selection of the ECOCart, and therefore the selection of a suitable acidic dialysis concentrate, is the responsibility of the treating physician. Treatment duration, frequency, and dosage must be determined by the treating physician.

Shelf life after dissolution

The liquid concentrate can be used for 2 years after the ECOCart has been dissolved, provided it is stored in the ECOMix storage tanks.

Disposal

ECOCards Original and ECOCards Revolution are the property of the manufacturer and must be returned to them.

ECOCards ToGo, if not returned to the manufacturer, must be disposed of in accordance with local regulations and laws. The ingredients of each ECOCart can be found on the product label. Before recycling the PE drum, the RFID tag must be drilled out of the coupling of the rising tube and disposed of separately as electronic waste. PE wrapping, the wooden pallet, and the strapping band must be disposed of separately.

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